# ACCU-CHEK® Safe-T-Pro Plus

03603539, 05217610

Sterile, single-use lancing device

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**EN** Sterile, single-use lancing device for obtaining capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.

Components of the single-use lancing device (see illustration ①)

- ① Description
- ② Release button
- ③ Housing
- ④ Penetration depth adjuster
- ⑤ Sterility cap of the needle
- ⑥ Penetration depth indicator
- ⑦ Low penetration depth (approx. 1.3 mm)
- ⑧ Medium penetration depth (approx. 1.8 mm, pre-set)
- ⑨ High penetration depth (approx. 2.3 mm)

**Preparing to obtain blood****2 Children younger than 1 year**

With children younger than 1 year of age, the capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole (see black areas in illustration ②).

The puncture must occur vertically to the sole (see arrows in the enlargement). While obtaining blood, the child should be held with its foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth of a maximum 42 °C.

The heel must be disinfected and then dried completely.

To obtain capillary blood from the heel of children under 1 year, use only the low or medium penetration depth. If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

**3 Children as of 1 year and adults**

We recommend obtaining capillary blood from the side of the fingertips, as these sites register the least amount of pain (see black areas in illustration ③). Any finger can be used for the puncture.

The patient's hands should be washed with warm water and soap, and then dried thoroughly. This ensures that the puncture site is clean and increases blood flow. If you are using alcohol wipes, the finger must also be completely dry.

**Obtaining blood****④ Removing the sterility cap**

► Twist the sterility cap and remove it.

⚠ Do not use the single-use lancing device if the sterility cap has been previously removed.

- In extreme cases, the needle can completely stick out of the housing so that injury cannot be completely excluded. Discard the defective single-use lancing device in accordance with your facility's disposal regulations.

**⑤ Setting the penetration depth**

► Turn the penetration depth adjuster to the desired penetration depth.

⚠ Do not set the penetration depth before removing the sterility cap.

• To obtain capillary blood from the heel of children under 1 year, use only the low or medium penetration depth. If the high penetration depth is used, the calcaneus (heel bone) is at risk of being injured.

**6 Obtaining blood**

- Hold the single-use lancing device between the index finger, middle finger and thumb.
- Press the single-use lancing device firmly against the chosen puncture site.
- Using your thumb, press the release button down completely.
- Gently massage the finger towards the fingertip or exert slight pressure on the heel to encourage a blood drop to form.
- For some diagnostic tests, the fingertip/heel must not be vigorously massaged or compressed, as doing so may produce incorrect test results.
- Always discard used single-use lancing devices in accordance with your facility's disposal regulations.

**Last update**

2019-04

**Accu-Chek Customer Care****United Kingdom**

Roche Diabetes Care Limited  
Charles Avenue, Burgess Hill  
West Sussex, RH15 9RY, United Kingdom  
Accu-Chek Customer Careline ①)  
UK Freephone number: 0800 701 000  
ROI Freephone number: 1 800 709 600  
① calls may be recorded for training purposes  
[www.accu-check.co.uk](http://www.accu-check.co.uk)  
[www.accu-check.ie](http://www.accu-check.ie)

**Malta**

Vivion Corporation  
Sanitas Building  
Tower Street, Msida MSD1824, Malta  
Freephone: 80073102

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© 2019 Roche Diabetes Care
ACCU-CHEK® en SAFE-T-PRO® are trademarks of Roche.